



Getting Started with Nipro Diagnostics and Microsoft® HealthVault™

Welcome!

This guide will show you how to set up your Nipro Diagnostics blood glucose monitor to upload your blood glucose results into your Microsoft HealthVault account.

With this guide, you will be able to:

1. **Set up** your blood glucose monitor and take a test reading
2. **Get ready** to use your blood glucose monitor with HealthVault
3. **Connect** your blood glucose monitor to your computer and upload your test reading to your HealthVault account

IMPORTANT: If you have not already created your HealthVault account and installed HealthVault Connection Center, please go to <http://healthvault.com/connectioncenter> and follow the instructions.

Let's get started!

Step 1: Set up your blood glucose monitor and take a test reading

- Open your Nipro Diagnostics blood glucose monitor packaging and remove the contents.
- Turn on your meter by pressing the "S" button on the top of meter and verify that the meter shows the correct date and time. If not, Set-up the date and time on your blood glucose monitor by following the instructions in your device owners booklet.
- Prepare your blood glucose monitor for glucose testing. Refer to the device owner's booklet for instructions on performing blood glucose testing.
- Insert a test strip into the monitor, once the apply sample symbol appears, apply a blood sample. The result of this blood glucose test will be the test reading used for setting up your device with HealthVault.

Step 2: Get ready to use your blood glucose monitor with HealthVault

- If the HealthVault Connection Center (HVCC) is not running on your computer, please start it and sign in to your account. Connection Center will then open to a screen showing your blood glucose monitoring device(s).
- If you do not see a picture of the Nipro Diagnostics blood glucose device in Connection Center, select **Set up a new device** and follow the on-screen instructions.
- Select the Nipro Diagnostics blood glucose monitor in the left hand panel of HealthVault Connection Center.

Step 3: Connect your blood glucose monitor to your computer and upload your test reading to HealthVault

- Plug the small end of your Nipro Diagnostics USB Smart Data Cable into the USB Port of your blood glucose monitor. If you are using a Nipro Diagnostics meter that uses a docking station and cable place your meter into the docking station with the cable attached.
- Plug the large end of your Nipro Diagnostics USB Smart Data Cable into a USB Port on your computer.

HealthVault Connection Center will know when your device has been connected to your computer and will prompt you to upload your test reading.

- Follow the on-screen prompts.

You will see a confirmation when your upload is complete. At this point you may unplug your blood glucose monitor or remove it from the docking station

To see your test results in your HealthVault account:

- Open your Internet browser and go to Healthvault.com (you will be asked to sign in if you haven't accessed your account in the past 20 minutes).
- Select your record.
- Click your **Health info** tab.

Whenever you have new test results on your device, you can follow these same steps to upload them to your HealthVault account to easily view results and track trends in your glucose results. For further help using your Nipro Diagnostics blood glucose monitor with HealthVault:

- Go to http://www.niprodiagnostics.com/downloads/niprodiagnostics_healthvault_troubleshootingguide.pdf.
- Refer to the owner's booklet that came with your blood glucose monitoring device.
- Call Nipro Diagnostics Customer Care Department at 1 (800) 803-6025.
- Go to http://www.niprodiagnostics.com/our_products/product_support.aspx