

Sidekick[®] Blood Glucose Monitoring System

What is the Sidekick[®] Blood Glucose Testing System?

Used to test blood glucose levels, the Sidekick[®] meter is the smallest all in one no coding disposable blood glucose testing system available, perfect for everyday use or travel. It's a vial with a built in meter!

What is no coding?

The Sidekick[®] no coding system does not require you to perform an extra step to match up your meter with your test strips. Just insert a strip and test.

What strips do I use for the Sidekick[®] meter?

The Sidekick[®] glucose test system is an all in one disposable system and includes 50 strips. Simply remove a test strip from the vial, insert it into the meter on top, and test. When you're out of test strips, or upon the expiration date, simply discard the unit. It's that simple.

How do I set up my Sidekick[®] meter?

No set up is required.

How do I calibrate my Sidekick[®] meter?

Calibration is not required. Use the strips included with the meter only.

How do I use the Sidekick[®] System?

See [instructional video](#) and refer to your [owner's booklet](#) included with your meter.

Can the Sidekick[®] System be used for alternate site testing?

Yes. Sidekick[®] is suitable for fingertip and forearm testing.

Do I need a lancet when testing with the Sidekick[®] meter?

Yes. A lancing device and sterile lancets are sold separately. We recommend the TRUEdraw™ lancing device for greater comfort and control to be used with our single-use sterile lancets.

I am on peritoneal dialysis, can I use this system?

Yes. The Sidekick[®] system is a great choice for you. Please consult your instructions for use for limitations.

Does the Sidekick[®] meter have memory? Does the meter store my results?

Yes. Sidekick[®] has a 50-test memory.

Can I upload my meter data into data management system?

No. The Sidekick[®] system offers on-the-go convenience. Some of our other TRUE brand meters offer upload capability.

What kind of batteries does the Sidekick[®] meter use?

Battery replacement is not necessary. The Sidekick[®] meter is intended to be used with the 50 test strips and then disposed.

Where can I find the Sidekick[®] System?

Our TRUE portfolio of products is available at your favorite retail store or mail service provider. [Where To Buy](#)