Scan this code with your smart phone for additional TRUE2go information. Always refer to the Owner's Booklet for detailed product information.

Meter, Test Strips & System Components

Top View

Front View

Test Port

Checking Your Blood Glucose Using TRUE2go®

A guide for testing your blood glucose

Simple Steps to Test Your Blood Glucose

Blood Glucose Testing

Strip Release Button

Test Port

Display

Test Strip

Contact End (TRUEtest™ name facing up)

Sample End (Tip)

1

Wash your hands with warm, soapy water.

2

Remove Test Strip from vial and close vial immediately. Insert Test Strip into Test Port with TRUEtest™ facing up. The Meter turns on.

3

Lance your finger.

4

Allow a blood drop to form, touch Tip of Strip to top of blood drop and allow blood to be drawn into Strip. Remove Test Strip Sample Tip from sample drop immediately after dashes appear across Meter Display.

CAUTION! Holding the Test Strip Sample Tip to the blood sample too long after the Meter begins testing may cause inaccurate results.

5

In as little as 4 seconds, glucose result will be displayed.

6

Hold Meter in a vertical position with Test Strip facing down. Press strip release button to discard the used Test Strip from the Meter.

WARNING!

NEVER reuse Test Strips. **NEVER** wipe test Strips with water, alcohol or any cleaner. **DO NOT** attempt to remove blood or control sample from Test Strips or clean Test Strips and re-use. Reuse of Test Strips WILL cause inaccurate results. **NEVER** add a second drop of sample to the Strip. Adding more sample gives an error message.

This guide provides a brief overview on testing your blood glucose. For more detailed information on complete blood glucose testing procedure, consult "Testing Your Blood" section in the Owner's Booklet. For assistance using your blood glucose System, call our Customer Care Department at 1-800-803-6025.

© 2011 Nipro Diagnostics, Inc. TRUE2go, TRUEtest and the Nipro Diagnostics logo are trademarks of Nipro Diagnostics, Inc. F4NPD08 Rev. 22

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